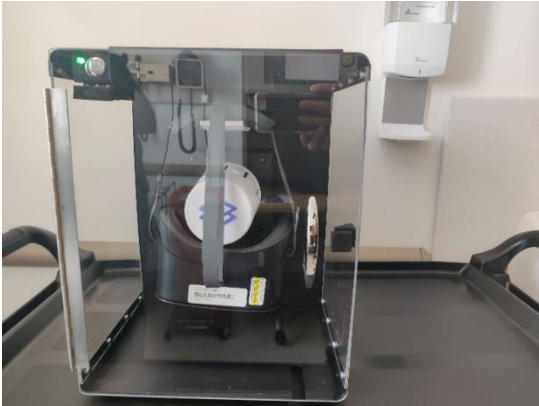
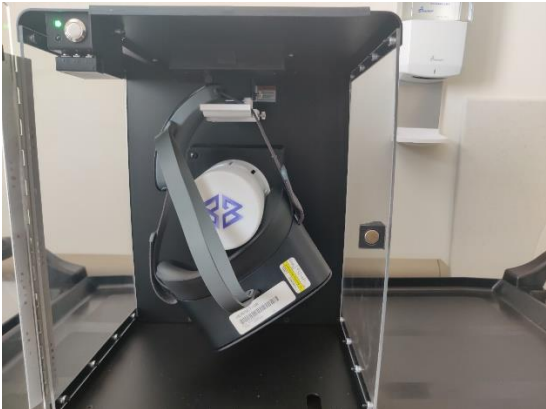


Disinfecting a Virtual Reality Headset with a CleanBox Device Standard Work

Last updated: 11/29/21	Owner: VASNHCS VR Program	Performed by: VR Session Host
Version: 1	Revised by: Original Version	Trigger: When there is a need to update the procedure
VA SNHCS Mission: Providing world class care and service to America's Heroes		
Standard Work Applicability: This Standard Work Applies when staff disinfect a Virtual Reality Headset with a CleanBox device.		

Standard Work Disinfecting a Virtual Reality Headset with a CleanBox Device

Major Steps		Details	Time / Frequency	Notes
1	ARRANGE FOR ACCESS TO USE THE CLEANBOX DEVICE		2 minutes / one time	•
2	DISINFECT THE HEADSET	<p>1. Cycle #1. Open door, place back of VR head strap on the white plate and make sure VR is centered on middle white cone. Close door and turn on red power switch on the back of the CleanBox. Press silver button on the top left of the CleanBox, next to the green light, to start first 60 second cycle; perform two cycles in this position. (See Figure 1)</p>  <p><i>Figure 1 VR Headset in CleanBox, positioned for Cycle #1.</i></p>	<p>1. 3 minutes / one time</p> <p>2. 3 minutes / one time</p>	Perform a total of 6 cycles to ensure every part of the headset is properly UV sanitized.

		<p>2. Cycle #2. When 1st cycle is complete, perform hand hygiene, open door and rotate headset at a 17-degree angle, shut the door and run 2 additional cycles.</p> <p>3. Cycle #3. Once both cycles are complete, perform hand hygiene, open door and rotate VR headset to the opposite side at a 17-degree angle and run 2 additional cycles. (See Figure 2)</p>  <p><i>Figure 2 VR Headset in CleanBox, positioned for Cycle #3.</i></p>	3. 3 minutes / one time	
3	ENSURE THAT THE CLEANBOX DEVICE IS ACCESSIBLE FOR FUTURE USE BY TEAM MEMBERS		2 minutes / one time	